

KOMFITILT

LATERAL TILT SYSTEM

EDITION 3.0

Provides offload and redistribution
of the users body weight;

Reducing & Preventing Pressure Areas
Helping Healing and Improving Comfort



JULY 2022

vikta®

01264 366 330
VIKTA.CO.UK

SPECIFICATIONS

Technically designed with your patient in mind.

Compatible with most mattress types including alternating air mattresses



Automated 10 Minute Cycle Time

Anti-slip matting on upper and lower surfaces to prevent movement



Anti-Slip Matting

Simple on/off control



Compatible with Profiling Beds

8 fastening straps to securely attach to bed frames



320kg Max User Weight

Fits on beds from 80cm wide up to 120cm wide (using the adjustable width extension)



3 Year Warranty

Uniquely developed to work with profiling beds.

SPECIFICATIONS
SPEICIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS

SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS

SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS

BENEFITS

Reduce the pressure and make a turn for the better.

Improved tissue viability for healing and prevention of pressure sores

Extends the time between turns due to Continuous Lateral Rotation Therapy (CLRT) (good results with patients with MS and MND)

Prevents pressure points developing at vulnerable sites

Better night's sleep equates to improved wellbeing

Continuous movement to promote improved blood and lymph flow

Less disturbance to husband/wife/partner/carer

Can speed up discharge from hospital as a less comprehensive home care package is required

Lower care costs due to less frequent care visits

Enables use of client's existing mattress or the use of a mattress from the Equipment Loan Store

Easy on/off controls for multiple carer situations

BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS

BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS

The lady is extremely comfortable with it and carers report that she is not as agitated during the night.

Occupational Therapist, Petersfield, Southern Health NHS Foundation Trust

BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS



FEATURES

The KomfiTilt system is placed on the bed frame under the mattress and is operated on an independent power and pump supply.

The KomfiTilt system gently tilts the user through a **left-neutral-right** cycle to relieve or prevent pressure points developing

FEATURES
FEATURES
FEATURES



Left

Neutral

Right

“My patient has for the first time in ages slept all night and does not notice the system turning her but knows that it is as when she does wake up he is in a different position. Her partner no longer needs to get up during the night to alter her position therefore she is getting an undisturbed full nights sleep. From the therapy perspective this equipment has met all goals set.”

Community Occupational Therapist, Havant & Waterlooville, Southern Health NHS Foundation Trust


Fits on any bed (underneath the mattress)

Can be used with foam and alternating air mattresses

Can be used with profiling beds (back angle adjustment, Trendelenburg / AntiTrendelenburg, knee break

Unique auto-deflate CPR function on switch off system deflates for patient transfer and personal care etc

Rental options available (minimum 10 day rental period)

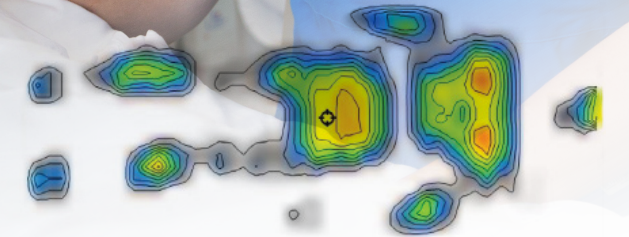

When switched ON the KomfiTilt system inflates to 20° to 25° to provide offload of pressure points and redistribution of body weight loadings.



EVEN MORE INTELLIGENCE
 EVEN MORE INTELLIGENCE
 EVEN MORE INTELLIGENCE
 EVEN MORE INTELLIGENCE

EVEN MORE INTELLIGENCE

We constantly evolve our products to stay at the forefront of technology and patient solutions.

Bluetooth Connection to App, monitor the KomfiTilt performance and make remote function adjustments.

Download and connect to your KomfiTilt via Bluetooth

Input Model No: (e.g. SQ-BT1955) this model number can be found on the pump box. When connected, a tracking record will show on the graph (the two colours represent the different cells).

Cycle Time adjustment




Press the 'Clock' icon and an audible alarm will sound and the green light will move to the next setting (cycle time can be adjusted between 10 / 15 / 20 / 25 minutes)

Cell Lock-Off


Press the 'Cell Tilt' icon and an audible alarm will sound and the green light will move to the next setting (B / A & B / A)

History Records

Press the 'History Records' and select from the list to show the stored record.

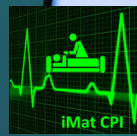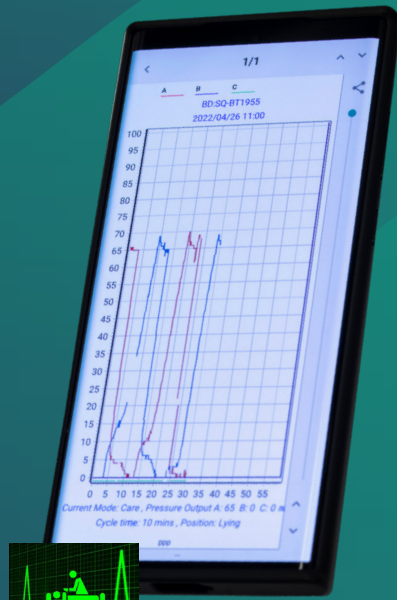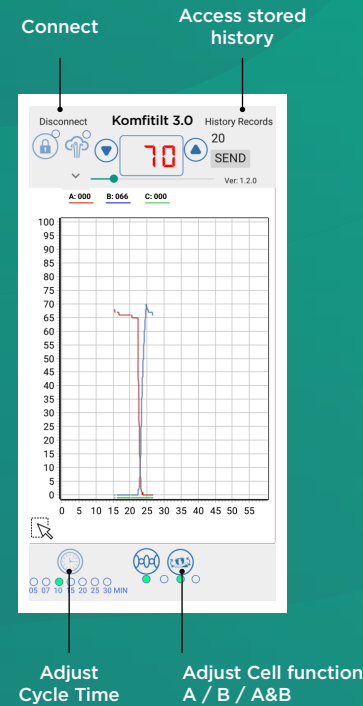

iMat IPC

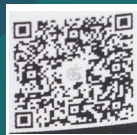

Download on the App Store



GET IT ON Google Play

Additional features of KomfiTilt Ed.3.0

www.vikta.co.uk

Integral pressure sensors for calibrated support

Audio & visual alarm for air-loss or low pressure fault

Audio & visual alarm for power supply failure


Modern compact design

Clear interface display

Normal ON/OFF function will activate Left-Supine-Right (A-B) function of the cells



Air Hose Connectors & Cpr Release

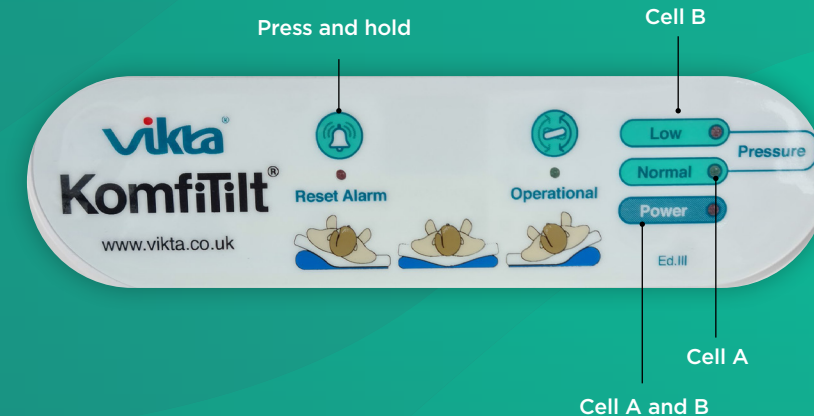

MANUAL LOCK-OFF OF LEFT OR RIGHT CELL

To Lock-Off either the Right or Left cell, so that it doesn't inflate during the operational cycle: On Start-Up, switch the power ON and the faceplate lights will light up.

Cell A only

Press the 'Reset Alarm' button and hold down. The 'Normal' green light will flash and the audible alarm will sound. Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell A only.

Cell B only

Press the 'Reset Alarm' button again and hold down. The 'Low' red light will flash and the audible alarm will sound. Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell B only.

Cell A & B in normal rotation

Press the 'Reset Alarm' button again and hold down. The 'Power' green light will flash and the audible alarm will sound. Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell A & B in the normal rotation.

EVEN MORE INTELLIGENCE
 EVEN MORE INTELLIGENCE
 EVEN MORE INTELLIGENCE



Great results with the Vikta KomfiTilt with a patient of mine who has a Grade 4 Sacral Pressure Sore that the Community Nurse had been attempting to heal for the past year.

My patient has MS is unable to alter her position in bed, contractures in lower limbs. The Vikta KomfiTilt was placed under her alternating mattress 8 weeks ago and now the sore has reduced in size and has healed.

The Community Nurses and the patients partner feel that the Vikta KomfiTilt is the reason for the great result and that the pressure sore should continue to fully heal.

Occupational Therapist



01264 366 330

VIKTA.CO.UK

Pressure Care Management
Unit 2 Edison Gate, West Portway, Andover SP10 3SE
www.pressurecaremanagement.com