

01264 366 330

VIKTA.CO.UK

EDITION V

KOMFITILT

LATERAL TILT SYSTEM

EDITION 5.0

vikta[®]

Provides offload and redistribution
of the users body weight;

**Reducing & Preventing Pressure Areas
Helping Healing & Improving Comfort**

FEATURES
FEATURES
FEATURES
FEATURES

FEATURES
FEATURES
FEATURES

FEATURES

HOW IT WORKS

The KomfiTilt system is placed on the bed frame under the mattress and is operated on an independent power and pump supply.



KomfiTilt

Pump


Fits on any bed
(underneath the mattress)

Can be used with foam and
alternating air mattresses

Can be used with
profiling beds (back angle
adjustment, Trendelenburg /
AntiTrendelenburg, knee break

Accommodates for bariatric
patients.

When switched ON the KomfiTilt system inflates
to 20° to 25° to provide offload of pressure points
and redistribution of body weight loadings.

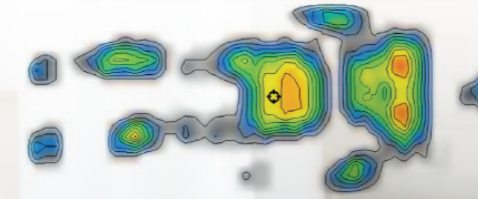

**Unique auto-deflate CPR
function on switch off system
deflates for patient transfer
and personal care etc**

Rental options available
(minimum 10 day rental period)



A RANGE OF MOVEMENT

The Edition 5 allows for a neutral function but if a patient only has sensitive areas on one side of their body, the focus for distributing the weight can be on that side of the body. This can work for a right tilt neutral/left tilt neutral or right neutral left motion



RIGHT



LEFT



NEUTRAL



SPECIFICATIONS

Technically designed with your patient in mind.

Compatible with most mattress types including alternating air mattresses



Automated Cycle Times (10/30/60min)

Anti-slip matting on upper and lower surfaces to prevent movement



Anti-Slip Matting

Simple on/off control



Compatible with Profiling Beds

8 fastening straps to securely attach to bed frames



320kg Max User Weight (for bariatric patients)

Fits on beds from 80cm wide up to 120cm wide (using the adjustable width extension)



3 Year Warranty

Uniquely developed to work with profiling beds



Advanced Tilt Features

Controlled faceplate on pump

SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS
SPECIFICATIONS SPECIFICATIONS

SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS
SPECIFICATIONS

BENEFITS

Reduce the pressure and make a turn for the better.

Improved tissue viability for healing and prevention of pressure sores

Extends the time between turns due to Continuous Lateral Rotation Therapy (CLRT) (good results with patients with MS and MND)

Prevents pressure points developing at vulnerable sites

Better night's sleep equates to improved wellbeing

Continuous movement to promote improved blood and lymph flow

Less disturbance to husband/wife/partner/carer

Can speed up discharge from hospital as a less comprehensive home care package is required

Lower care costs due to less frequent care visits

Enables use of client's existing mattress or the use of a mattress from the Equipment Loan Store

Easy on/off controls for multiple carer situations

BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS

BENEFITS
BENEFITS
BENEFITS
BENEFITS
BENEFITS

The lady is extremely comfortable with it and carers report that she is not as agitated during the night.

Occupational Therapist, Petersfield, Southern Health NHS Foundation Trust



BENEFITS
BENEFITS
BENEFITS

KOMFITILT 5.0 IS EVEN

MORE BESPOKE MORE INTELLIGENT

We constantly evolve our products to stay at the forefront of technology and patient solutions.


Bluetooth Connection to App, monitor the KomfiTilt performance and make remote function adjustments.

Download and connect to your KomfiTilt via Bluetooth
Input Model No: (e.g. SQ-BT1955) this model number can be found on the pump box. When connected, a tracking record will show on the graph (the two colours represent the different cells).

Cycle Time adjustment
Press the 'Clock' icon and an audible alarm will sound and the green light will move to the next setting (cycle time can be adjusted between 10 / 15 / 20 / 25 minutes)

Cell Lock-Off
Press the 'Cell Tilt' icon and an audible alarm will sound and the green light will move to the next setting (B / A & B / A)

History Records
Press the 'History Records' and select from the list to show the stored record.

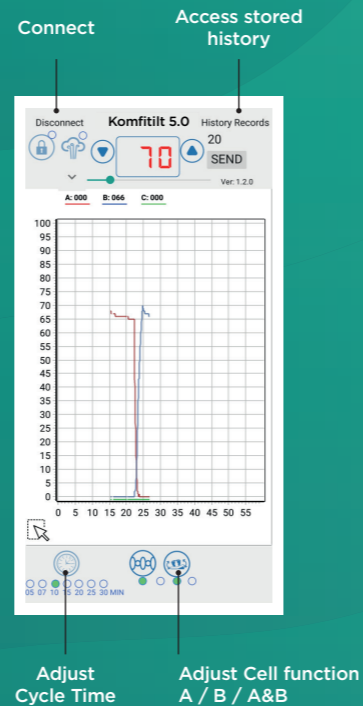

NEW features of KomfiTilt Ed.5.0

The cycle time settings and tilt function can be changed simply and easily by pressing the buttons on the front faceplate panel, by a carer or healthcare professional as well as via our bluetooth connection app

This enables the KomfiTilt to be set up to suit the patients individual requirements

Cycle times: (10/30/60min)

Additional features of KomfiTilt Ed.5.0

Integral pressure sensors for calibrated support

Audio & visual alarm for air-loss or low pressure fault

Audio & visual alarm for power supply failure

Modern compact design

Clear pump interface display with optional tilt function settings:

- Option 1: Left cell neutral left cell
- Option 2: Left cell neutral right cell
- Option 3: Right cell neutral right cell



CPR cord for rapid response deflation of cells




Normal ON/OFF function will activate Left-Supine-Right (A-B) function of the cells



Air Hose Connectors & CPR Release

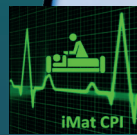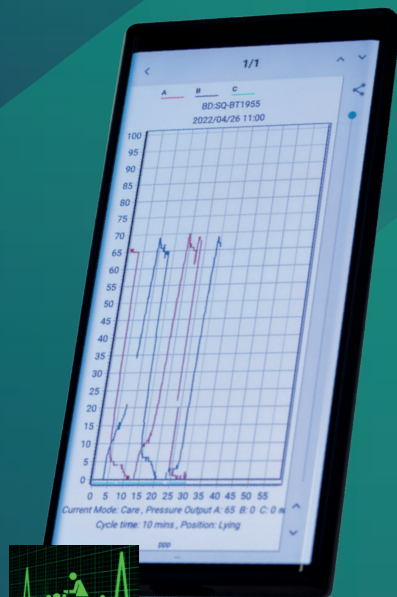


iMat IPC

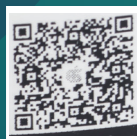

Download on the App Store



GET IT ON Google Play

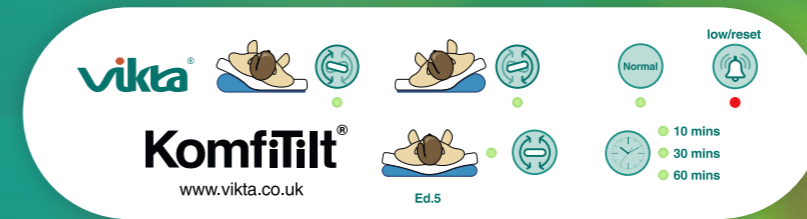

EVEN MORE BESPOKE EVEN MORE INTELLIGENCE



Great results with the Vikta KomfiTilt with a patient of mine who has a Grade 4 Sacral Pressure Sore that the Community Nurse had been attempting to heal for the past year.

My patient has MS is unable to alter her position in bed, contractures in lower limbs. The Vikta KomfiTilt was placed under her alternating mattress 8 weeks ago and now the sore has reduced in size and has healed.

The Community Nurses and the patients partner feel that the Vikta KomfiTilt is the reason for the great result and that the pressure sore should continue to fully heal.

Occupational Therapist



01264 366 330

VIKTA.CO.UK

Pressure Care Management
Unit 2 Edison Gate, West Portway, Andover SP10 3SE
www.pressurecaremanagement.com